


IN THE BOX



Battrode™
(packaged inside
CAM and Battrode
pouch)

Recorder
(packaged inside
CAM box)

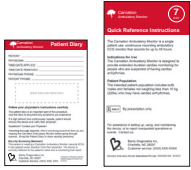

**Patient Diary and
Quick Reference
Instructions**
(inside CAM and
Battrode box)

Carnation Ambulatory Monitor
Quick Reference Instructions DWG000137G 04/2019



The Carnation Ambulatory Monitor is a continuously recording ambulatory ECG monitor that records for up to 7 days.

Indications for Use

The Carnation Ambulatory Monitor is designed to provide extended duration cardiac monitoring for people who are suspected of having cardiac arrhythmias.

Patient Population

The intended patient population includes both males and females not weighing less than 10 kg (22lbs) who may have cardiac arrhythmias.



By prescription only

OPERATIONAL INSTRUCTIONS

If the Carnation Ambulatory Monitor becomes soiled, patients may gently wipe exterior of device, using a clean, dry cloth.

If Battrode becomes loose or detached from skin, patient should press the adhesive portions of monitor back in place.

If a high pitch tone continuously repeats, patient should remove the device and notify doctor.

TROUBLESHOOTING

See Operational Instructions above OR
Visit www.bardyd.com for more information.

FAQS

If you have questions regarding the Carnation Ambulatory monitor, visit our website:

www.bardyd.com

⚠ CAUTION STATEMENTS

CAUTION: Proper skin prep required to achieve 7 days wear.

CAUTION: Touching the adhesive can reduce adhesive performance. When peeling the liner from the back of the Battrode, hold onto the tabs at its ends.

CAUTION: Inability to diagnose if patient and physician data is missing. The patient diary card must be kept with the Carnation Ambulatory Monitor to ensure proper diagnosis.

For assistance in setting up, using, and maintaining the device, or to report unexpected operations or events, contact us:



Bardy Diagnostics Inc.

US Customer service: (844) 422-7393
EU Customer service: +31 (0) 46 7630422



MPS Medical Product Service GmbH
Borggasse 20
35619 Braunfels
Germany




IN THE BOX



Battrode™
(packaged inside
CAM and Battrode
pouch)

Recorder
(packaged inside
CAM box)

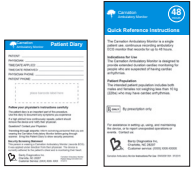

Patient Diary and
Quick Reference
Instructions
(inside CAM and
Battrode box)

The Carnation Ambulatory Monitor is a continuously recording ambulatory ECG monitor that records for up to 48 hours.

Indications for Use

The Carnation Ambulatory Monitor is designed to provide extended duration cardiac monitoring for people who are suspected of having cardiac arrhythmias.

Patient Population

The intended patient population includes both males and females not weighing less than 10 kg (22lbs) who may have cardiac arrhythmias.



By prescription only

OPERATIONAL INSTRUCTIONS

If the Carnation Ambulatory Monitor becomes soiled, patients may gently wipe exterior of device, using a clean, dry cloth.

If Battrode becomes loose or detached from skin, patient should press the adhesive portions of monitor back in place.

If a high pitch tone continuously repeats, patient should remove the device and notify doctor.

TROUBLESHOOTING

See Operational Instructions above OR
Visit www.bardyd.com for more information.

FAQS

If you have questions regarding the Carnation Ambulatory monitor, visit our website:

www.bardyd.com

Carnation Ambulatory Monitor
Quick Reference Instructions DWG000136H 04/2019



⚠ CAUTION STATEMENTS

CAUTION: Proper skin prep required to achieve 48 hours wear.

CAUTION: Touching the adhesive can reduce adhesive performance. When peeling the liner from the back of the Battrode, hold onto the tabs at its ends.

CAUTION: Inability to diagnose if patient and physician data is missing. The patient diary card must be kept with the Carnation Ambulatory Monitor to ensure proper diagnosis.

For assistance in setting up, using, and maintaining the device, or to report unexpected operations or events, contact us:



Bardy Diagnostics Inc.

US Customer service: (844) 422-7393
EU Customer service: +31 (0) 46 7630422



MPS Medical Product Service GmbH
Borggasse 20
35619 Braunfels
Germany

