

Baxter

Adhesive Ring

The Adhesive Ring is intended to help secure the **CAM** Patch to the patient's skin during the prescribed wear period.

INSTRUCTIONS FOR USE


Follow the instructions below if your healthcare provider gave you an Adhesive Ring to apply with your **CAM** Patch device. Please review this guide completely to ensure you understand all the steps and the order in which they must be performed.

INDICATION FOR USE: For use with **Bardy Diagnostics Carnation Ambulatory Monitor (CAM)**.

CAUTION: Skin irritation or allergic reaction – In case of skin irritation or allergic reaction, contact your healthcare provider.

Understanding the Adhesive Ring


The following illustration shows the three Adhesive Ring layers.

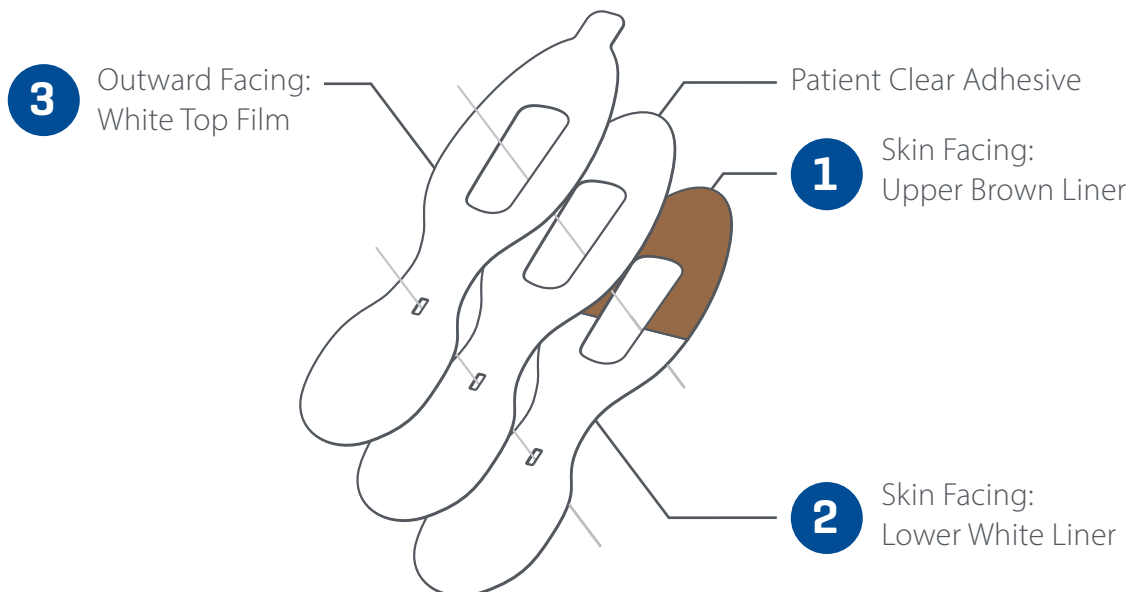


How to Apply the Adhesive Ring

NOTE: Do not use if opened or damaged.

- 1.** Position the Adhesive Ring over the applied **CAM** Patch, with the white top film facing outward.

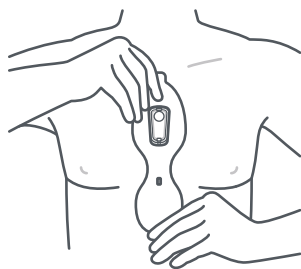

- 2.** Gently peel off the upper brown liner [**1**] tab upwards. Press and rub to secure adhesion.

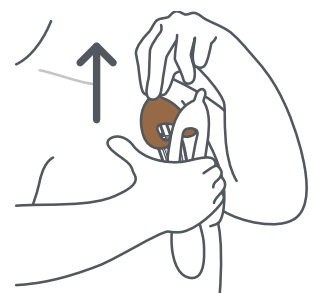

- 3.** Gently peel off the lower white liner [**2**] downwards. Press and rub the Adhesive Ring to secure adhesion.




- 4.** Gently peel off the white top film [**3**] tab downwards and rub the Adhesive Ring around the **CAM** Patch.




NOTE: When done applying, the Adhesive Ring will be translucent.

How to Remove the Adhesive Ring


1. Wipe border of the Adhesive Ring with the Adhesive Remover Wipe.

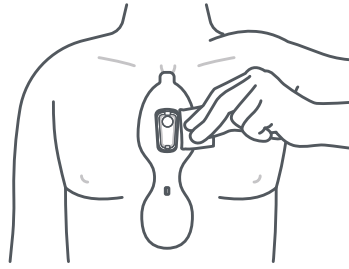

2. Slowly peel downward while applying adhesive remover along the edge of skin and underside of Adhesive Ring and CAM Patch.

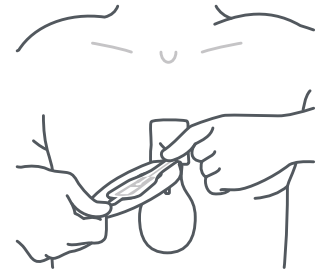

3. Place the used CAM Patch with the Adhesive Ring on the outside of its original pouch.

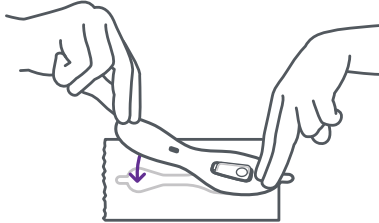









4. Wipe off any remaining adhesive on the skin.

5. For returning the CAM Patch follow the healthcare provider directions.

NOTE: Place the CAM Patch on the outline area.

Symbols

Bardy Diagnostics products display one or more of these symbols.

SYMBOL	DESCRIPTION	SYMBOL	DESCRIPTION
	Catalogue Number		Manufacturer
	Do Not Re-Use		Date of Manufacture
	Unique Device Identifier		Use-By-Date
	Prescription Use Only		Batch Code

Support

For assistance in applying or using the Adhesive Ring, or to report unexpected events.

CONTACT US:



Bardy Diagnostics, Inc.
 220 120th Avenue NE, Ste 100
 Bellevue, WA 98005
 USA
 US Customer Service: 1-844-777-9283